


FLEXIMATIC APPLICATOR

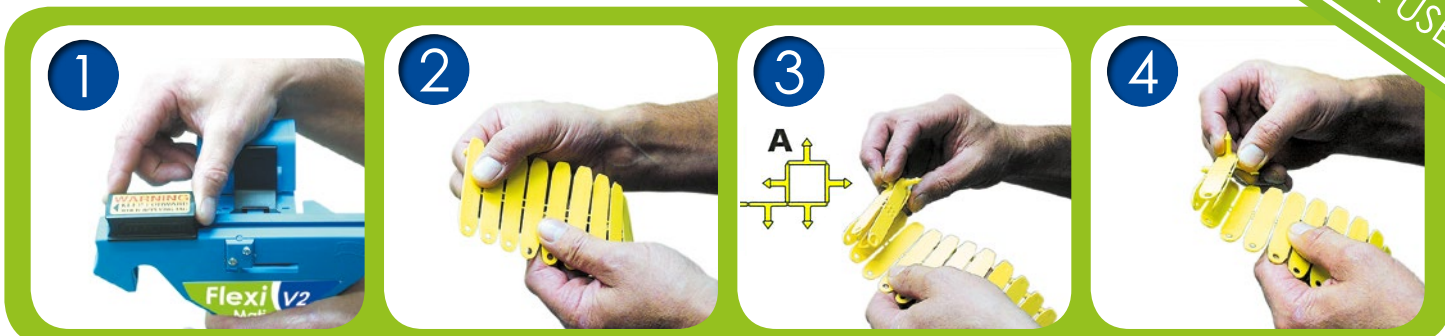

1 Ensure loader/cutter is in locked position (at front of applicator).

2 Hold FlexiTags as shown, holes to the left, pins downward.

3 When beginning roll, make sure there is a gap between the first four FlexiTags (Fig A).

4 Roll FlexiTags up firmly. Bias the pins outwards to avoid jamming.



5 Leave the last two FlexiTags unrolled.

6 Slide the rolled FlexiTags into the drum.

7 Depress the clamp firmly. Slide tags out until location lugs are between first and second tag.

8 Squeeze lever and keep closed. The FlexiTag is in cutting position. Slide to the loader/cutter fully back. Tag is now cut.



9 Firmly slide loader to forward locked position. The FlexiTag is now loaded.

10 Release lever, the FlexiMatic is now ready to apply the FlexiTag.

11 Select optimum position on ear for tag. Squeeze lever firmly (keep lever closed). The FlexiTag is now applied.

12 With the lever still closed repeat instructions 8 onwards.



Wash regularly in clean water

Clean after use. Submerge in hot soapy water and wash. Position to drain as per picture to the right.



Contact our friendly Customer Service Team

1300 138 247

www.allflex.com.au



Identify the Difference